

## DRESSDERM BLISTER PLASTER

DRESSDERM® HYDROCOLLOID MIXED BLISTER PLASTERS CONTAINS BLISTER PLASTERS IN VARIOUS SIZES TO SUIT BLISTERS ON DIFFERENT PARTS OF THE FOOT.

BLISTER PLASTER IS A GEL WITH MOISTURE ABSORBING PARTICLES. THICK CUSHIONING REDISTRIBUTES PRESSURE FROM THE BLISTER TO DELIVER INSTANT PAIN RELIEF.

### DESCRIPTION:

DRESSDERM® HYDROCOLLOID BLISTER PLASTERS

### FEATURES:

- DRESSDERM® HYDROCOLLOID BLISTER PLASTERS ULTRA ADHESIVE COMES WITH TAPERED EDGES SO IT FLEXES WITH MOVEMENT AND STAYS IN PLACE FOR SEVERAL DAYS
- INSTANT PAIN RELIEF
- FAST HEALING
- STAYS IN PLACE FOR DAYS
- DEEP CUSHIONING TO RELIEVE PRESSURE AND PREVENT PAINFUL RUBBING
- TRANSLUCENT EDGES BLEND IN WITH THE SKIN
- FORM FITTING WATERPROOF AND BREATHABLE DESIGN KEEPS NATURAL MOISTURE IN AND GERMS OUT FOR FASTER NATURAL HEALING

### DIRECTIONS:

- CLEAN AND DRY YOUR SKIN BEFORE USE, ENSURING BLISTER AREA IS FREE OF CREAMS AND OILS.
- REMOVE THE BOTTOM PAPER, AVOIDING TOUCHING THE ADHESIVE SIDE.
- APPLY PLASTER FIRMLY OVER BLISTER, ENSURING THE EDGES ARE WELL SMOOTHED DOWN.
- LEAVE IN PLACE UNTIL IT STARTS TO DETACH (NOTE: MAY STAY IN PLACE UP TO SEVERAL DAYS).
- TO REMOVE PLASTER: DO NOT PULL UPWARDS BUT SLOWLY STRETCH ALONG THE SKIN.

### STORAGE:

- KEEP AWAY FROM DIRECT SUNLIGHT
- STORE BELOW 25°C
- KEEP AWAY FROM WATER

### SHELF LIFE:

- 3 YEARS



### SIZE

4.2 cm X 6.8 cm

2 cm X 6 cm

1.7 cm X 6 cm

\*Other sizes available on request

